

RECOGNIZE POTENTIAL HAZARDS OF WEARING AN N95

VISIT aub.ie/n95help FOR LATEST RESOURCES AND RECOMMENDATIONS

VERIFY YOU ARE USING A NIOSH-APPROVED RESPIRATOR

YOU CAN FIND A LIST OF APPROVED N95 RESPIRATORS ON THE NIOSH WEBSITE.

USE THE RESPIRATOR SAFELY AND EFFECTIVELY

REFER TO THE MANUFACTURER'S INSTRUCTIONS THAT WERE INCLUDED WITH THE RESPIRATOR.

KNOW HOW TO DON AND DOFF YOUR SPECIFIC RESPIRATOR

FOR MORE INFORMATION, VISIT THE NIOSH WEBSITE. www.cdc.gov/niosh/

CONDUCT A USER SEAL CHECK BEFORE EACH USE

SEAL CHECK INSTRUCTIONS DIFFER FOR EACH MANUFACTURER. READ ALL INSTRUCTIONS BEFORE USE.

FILL OUT THE VOLUNTARY USE AUTHORIZATION FORM

VISIT aub.ie/n95voluntary OR SCAN QR CODE AT THE BOTTOM ON THIS DOCUMENT.

IF YOU PLAN TO REUSE A N95 FILTERING FACEPIECE RESPIRATOR PLEASE BE ADVISED:

THERE IS NO WAY TO DETERMINE THE MAXIMUM POSSIBLE NUMBER OF SAFE REUSES OF AN N95 RESPIRATOR THAT WOULD APPLY IN ALL CASES.

THESE RECOMMENDATIONS ARE DESIGNED TO PROVIDE PRACTICAL ADVICE SO THAT N95 RESPIRATORS ARE DISCARDED BEFORE THEY BECOME A SIGNIFICANT RISK FOR CONTACT TRANSMISSION, OR THEIR FUNCTIONALITY IS REDUCED.

ALWAYS PERFORM SEAL CHECK BEFORE EACH USE



FOLLOW MANUFACTURER RECOMMENDATIONS FOR PERFORMING SEAL CHECK/



ALWAYS WEAR GLOVES WHEN PERFORMING A SEAL CHECK.



IF AIR LEAKS OUT BETWEEN FACE AND THE RESPIRATOR, THE RESPIRATOR **DOES NOT FIT** PROPERLY.

DO NOT USE WITHOUT PROPER FIT.

HOW TO STORE YOUR N95 RESPIRATOR

STORE IN CLEAN, BREATHABLE CONTAINER (SUCH AS A PAPER BAG)

- STORE AWAY FROM:
- SUNLIGHT
 - DUST
 - EXTREME TEMPERATURES
 - MOISTURE
 - DAMAGING CHEMICALS



WHEN TO DISPOSE OF YOUR N95 RESPIRATOR

- AFTER AEROSOL GENERATING OPERATIONS
- IF EXTERNALLY CONTAMINATED WITH FLUID
- IF INTEGRITY OF ANY PART IS COMPROMISED
- IF UNABLE TO ACHIEVE PROPER SEAL



SCAN QR CODE TO COMPLETE VOLUNTARY USE AUTHORIZATION FORM

